

## Параметры для ввода в программу анализатора Olympus AU 400/640

Specific Test Parameters	
General	LIH ISE Range
Test Name:	Phos < > Type: Serum Operation: Yes
Sample:	Volume: 2.5 µL Dilution: 0 µL Pre-Dilution Rate: 1
Reagents:	R1 Volume: 200 µL Dilution: 0 µL Min OD Max OD
	R2 Volume: 50 µL Dilution: 0 µL L: -0.1 H: 2.5
Wavelength:	Pri: 340 Sec: 540 Reagent OD limit: First L: -0.1 First H: 2.5
Method:	END Last L: -0.1 Last H: 2.5
Reaction slope:	+
Measuring Point 1:	First: 0 Last: 27 Dynamic Range: L: 0.06 H: 9.69
Measuring Point 2:	First: 0 Last: 10 Correlation Factor: A: 1 B: 0
Linearity:	%
No Lag Time:	On-board stability period: 999

Specific Test Parameters	
General	LIH ISE Range
Test Name:	Phos < > Type: Serum
Value/Flag:	Level L: Level H:
Normal Ranges:	Age L Age H L H
1.	Sex Year Month Year Month L H
2.	Sex Year Month Year Month L H
3.	Sex Year Month Year Month L H
4.	Sex Year Month Year Month L H
5.	Sex Year Month Year Month L H
6.	Sex Year Month Year Month L H
7. None Selected	0.85 1.95
8. Out of Range	L H
Panic Value:	Unit: mmol/L Decimal places: 2

Calibration Specific	
General	ISE
Test Name:	Phos < > Type: Serum
Calibration Type:	AB Formula: Y=AX+B Counts: 3 Process: CONC
Point 1:	Cal. No. * OD CONC ** Factor/OD-L -99999 Factor/OD-H 99999
Point 2:	Cal. No. OD CONC Factor/OD-L Factor/OD-H
Point 3:	Cal. No. OD CONC Factor/OD-L Factor/OD-H
Point 4:	Cal. No. OD CONC Factor/OD-L Factor/OD-H
Point 5:	Cal. No. OD CONC Factor/OD-L Factor/OD-H
Point 6:	Cal. No. OD CONC Factor/OD-L Factor/OD-H
Point 7:	Cal. No. OD CONC Factor/OD-L Factor/OD-H
1-Point Cal. Point:	With CONC-0 Slope Check: None Advanced Calibration:
MB Type Factor:	Calibration Stability Period: 999

Диапазон нормальных значений указан в соответствии с рекомендациями производителя реагентов к набору и может быть изменен пользователем в соответствии с местными требованиями.

Калибровка линейная, АВ (калибратор TruCal U, или стандарт из набора).

\*\* -вводится из паспорта к калибратору (стандарт)

Контроль TruLab N и TruLab P.

\*-вводится в соответствии с настройками системы