

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **aAmy** Test Code: IL_ **aAmy-d** Measure Sample Reagent Ranges Limits Calibration

Sample Type
Serum

Reaction Cycle
Standard Extended

Reporting Unit
Ед/л * Decimal Points: 1
User Define:
Conv. Factor: 0.000

Methodology
Type: End Point Rate
Measuring Point: 22 - 31
Photometric: 1 Wavelength 2 Wavelength
Primary: 405 Secondary: 546
Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN **aAmy** Test Code: IL_ **aAmy-d** Measure Sample Reagent Ranges Limits Calibration

Sample Volume

	Sample		Dilution	
	Volume	Sample Vol.	Diluent Vol.	
1	3.0	0.0	0.0	
2	3.0	10.0	90.0	
3	6.0	0.0	0.0	
4	2.0	5.0	95.0	

Diluent:
Diluent Warning Limit: 0 Tests

Reagent Volume

R1: aAmy-d1 R2: aAmy-d2

	Volume	Diluent Vol.	stiring	Warning Limit(tests)	Stability(days)
R1	140	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	35	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

Condition No.	1	2	3	4
First Run	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Samp.Vol.Reduction	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Below N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Above N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic H	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
User Range L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> User Range H	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Noise	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Prozone	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> HIGH!	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> ABS!	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

альфа-Амилаза

IFCC, PNP

Ranges			Limits																																																																																				
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0</td> <td>100</td> </tr> <tr> <td>Female</td> <td>0</td> <td>100</td> </tr> <tr> <td>Other</td> <td>0</td> <td>100</td> </tr> </tbody> </table>			Normal Range				Lower	Upper	Male	0	100	Female	0	100	Other	0	100	<table border="1"> <thead> <tr> <th>Reaction Slope</th> <th>Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Negative <input checked="" type="radio"/> Positive </td> <td> <input checked="" type="radio"/> Above <input type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>25 %</td> </tr> </tbody> </table> </td> <td> <table border="1"> <tbody> <tr> <td>2500.0 mAbs</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>2000</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>2000</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> </tbody> </table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>25 %</td> </tr> </tbody> </table>	Non-Linear Limit	25 %	<table border="1"> <tbody> <tr> <td>2500.0 mAbs</td> </tr> </tbody> </table>	2500.0 mAbs	<table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0	<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>2000</td> </tr> </tbody> </table>			User Range		Lower	Upper	2	2000			<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>2000</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	2	2000			<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>			Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0			<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>			Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off		
Normal Range																																																																																							
	Lower	Upper																																																																																					
Male	0	100																																																																																					
Female	0	100																																																																																					
Other	0	100																																																																																					
Reaction Slope	Absorbance Limit																																																																																						
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																																																																						
<table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>25 %</td> </tr> </tbody> </table>	Non-Linear Limit	25 %	<table border="1"> <tbody> <tr> <td>2500.0 mAbs</td> </tr> </tbody> </table>	2500.0 mAbs																																																																																			
Non-Linear Limit																																																																																							
25 %																																																																																							
2500.0 mAbs																																																																																							
<table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0																																																																											
Prozone Limit																																																																																							
<input type="radio"/> Above <input checked="" type="radio"/> Below																																																																																							
<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none																																																																																			
Limit	Equation																																																																																						
0.0	none																																																																																						
<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0																																																																																				
Judge Point																																																																																							
0																																																																																							
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>2000</td> </tr> </tbody> </table>			User Range		Lower	Upper	2	2000																																																																															
User Range																																																																																							
Lower	Upper																																																																																						
2	2000																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>2000</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	2	2000																																																																															
ValidRange																																																																																							
Lower	Upper																																																																																						
2	2000																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>			Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																																																											
Serum Index Limits																																																																																							
	Limit																																																																																						
Hemolysis	0.0																																																																																						
Icterus	0.0																																																																																						
Lipemia	0.0																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>			Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																	
Qualitative																																																																																							
<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																						

Test Name	Test Code	Measure	Sample Reagent	Ranges Limits	Calibration																				
NN	aAmy	IL_	aAmy-d																						
Calibration		Condition		Calibration/R-Blank Limit																					
<table border="1"> <thead> <tr> <th>Method</th> <th>Curve Type</th> </tr> </thead> <tbody> <tr> <td>1-point</td> <td>Linear</td> </tr> <tr> <th>Repeats</th> <th>Stability</th> </tr> <tr> <td>2</td> <td>days</td> </tr> <tr> <td>No</td> <td></td> </tr> </tbody> </table>		Method	Curve Type	1-point	Linear	Repeats	Stability	2	days	No		<input checked="" type="checkbox"/> Reagent Blank <input type="checkbox"/> Auto Reagent Blank by Bottle <input type="checkbox"/> Ask for calibration when reagent lot changes		<table border="1"> <tbody> <tr> <td>R-Blank Limit</td> <td>2500.0 mAbs</td> </tr> <tr> <td>Cal Reps Range</td> <td>20.0 %</td> </tr> <tr> <td>Min Cal Reps</td> <td>50.0 mAbs</td> </tr> <tr> <td>Factor Change</td> <td>20 %</td> </tr> <tr> <td>M-Point Curve Fit</td> <td>0.0 %</td> </tr> </tbody> </table>		R-Blank Limit	2500.0 mAbs	Cal Reps Range	20.0 %	Min Cal Reps	50.0 mAbs	Factor Change	20 %	M-Point Curve Fit	0.0 %
Method	Curve Type																								
1-point	Linear																								
Repeats	Stability																								
2	days																								
No																									
R-Blank Limit	2500.0 mAbs																								
Cal Reps Range	20.0 %																								
Min Cal Reps	50.0 mAbs																								
Factor Change	20 %																								
M-Point Curve Fit	0.0 %																								
<table border="1"> <thead> <tr> <th colspan="2">Calibrator</th> </tr> <tr> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>TruCalU **</td> </tr> <tr> <td>2</td> <td></td> </tr> <tr> <td>3</td> <td></td> </tr> <tr> <td>4</td> <td></td> </tr> <tr> <td>5</td> <td></td> </tr> </tbody> </table>		Calibrator		Calibrator	Conc	1	TruCalU **	2		3		4		5											
Calibrator																									
Calibrator	Conc																								
1	TruCalU **																								
2																									
3																									
4																									
5																									

альфа-Амилаза

IFCC, PNP



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU) или по фактору (см инструкцию).
Контроль по TruLab N и TruLab P.