

# АЛТ (аланинаминотрансфераза)

IFCC, without P5P (метод без перидоксальфосфата)



## Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN ALT    Test Code: IL\_ ALT-d    Measure    Sample Reagent    Ranges Limits    Calibration

Sample Type: Serum

Reaction Cycle: Standard    Extended

Reporting Unit: Ед/л \*    Decimal Points: 1    User Define:    Conv. Factor: 0.000

Methodology: Type: Rate    Measuring Point: 19 - 29    Photometric: 2 Wavelength    Primary: 340    Secondary: 546    Correction Constant: Slope: 1.000    Intercept: 0.000

Test Name: NN ALT    Test Code: IL\_ ALT-d    Measure    Sample Reagent    Ranges Limits    Calibration

Sample Volume:    Sampling Condition:

Sample	Dilution	Diluent
Volume	Sample Vol.	Diluent Vol.
1	14.0    0.0	0.0
2	2.0    0.0	0.0
3	25.0    0.0	0.0
4	10.0    5.0	95.0

Diluent Warning Limit: 0 Tests

Reagent Volume: R1 ALT-d1    R2 ALT-d2

	Volume	Diluent Vol.	stiring	Warning Limit(tests)	Stability(days)
R1	140	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	35	0.0	<input checked="" type="checkbox"/>	20	No Control

Condition No.	1	2	3	4
First Run	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Samp.Vol.Reduction	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Below N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Above N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic H	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
User Range L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
User Range H	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Noise	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Prozone	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
HIGH!	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
ABS!	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

# АЛТ (аланинаминотрансфераза)

IFCC, without P5P (метод без перидоксальфосфата)

Ranges			Limits																																																																																
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0</td> <td>41</td> </tr> <tr> <td>Female</td> <td>0</td> <td>31</td> </tr> <tr> <td>Other</td> <td>0</td> <td>41</td> </tr> </tbody> </table>			Normal Range				Lower	Upper	Male	0	41	Female	0	31	Other	0	41	<table border="1"> <thead> <tr> <th>Reaction Slope</th> <th>Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td> <input checked="" type="radio"/> Negative                             <input type="radio"/> Positive                         </td> <td> <input type="radio"/> Above                             <input checked="" type="radio"/> Below                         </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>25 %</td> </tr> </tbody> </table> </td> <td> <table border="1"> <tbody> <tr> <td>700.0 mAbs</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above                             <input checked="" type="radio"/> Below                         </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>600</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>600</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3">                     Qualitative  <input type="radio"/> On                     <input checked="" type="radio"/> Off                 </td> <td colspan="2"></td> </tr> </tbody> </table>		Reaction Slope	Absorbance Limit	<input checked="" type="radio"/> Negative <input type="radio"/> Positive	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>25 %</td> </tr> </tbody> </table>	Non-Linear Limit	25 %	<table border="1"> <tbody> <tr> <td>700.0 mAbs</td> </tr> </tbody> </table>	700.0 mAbs	<table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above                             <input checked="" type="radio"/> Below                         </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0	<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>600</td> </tr> </tbody> </table>			User Range		Lower	Upper	2	600			<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>600</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	2	600			<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>			Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0			Qualitative <input type="radio"/> On <input checked="" type="radio"/> Off				
Normal Range																																																																																			
	Lower	Upper																																																																																	
Male	0	41																																																																																	
Female	0	31																																																																																	
Other	0	41																																																																																	
Reaction Slope	Absorbance Limit																																																																																		
<input checked="" type="radio"/> Negative <input type="radio"/> Positive	<input type="radio"/> Above <input checked="" type="radio"/> Below																																																																																		
<table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>25 %</td> </tr> </tbody> </table>	Non-Linear Limit	25 %	<table border="1"> <tbody> <tr> <td>700.0 mAbs</td> </tr> </tbody> </table>	700.0 mAbs																																																																															
Non-Linear Limit																																																																																			
25 %																																																																																			
700.0 mAbs																																																																																			
<table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above                             <input checked="" type="radio"/> Below                         </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0																																																																							
Prozone Limit																																																																																			
<input type="radio"/> Above <input checked="" type="radio"/> Below																																																																																			
<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none																																																																															
Limit	Equation																																																																																		
0.0	none																																																																																		
<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0																																																																																
Judge Point																																																																																			
0																																																																																			
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>600</td> </tr> </tbody> </table>			User Range		Lower	Upper	2	600																																																																											
User Range																																																																																			
Lower	Upper																																																																																		
2	600																																																																																		
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>600</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	2	600																																																																											
ValidRange																																																																																			
Lower	Upper																																																																																		
2	600																																																																																		
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>			Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																																																							
Serum Index Limits																																																																																			
	Limit																																																																																		
Hemolysis	0.0																																																																																		
Icterus	0.0																																																																																		
Lipemia	0.0																																																																																		
Qualitative <input type="radio"/> On <input checked="" type="radio"/> Off																																																																																			

Test Name	Test Code	Measure	Sample Reagent	Ranges Limits	Calibration														
NN	ALT	IL_	ALT-d																
Calibration		Condition		Calibration/R-Blank Limit															
<table border="1"> <thead> <tr> <th>Method</th> <th>Curve Type</th> </tr> </thead> <tbody> <tr> <td>1-point</td> <td>Linear</td> </tr> </tbody> </table>		Method	Curve Type	1-point	Linear	<input checked="" type="checkbox"/> Reagent Blank <input type="checkbox"/> Auto Reagent Blank by Bottle <input type="checkbox"/> Ask for calibration when reagent lot changes		<table border="1"> <tbody> <tr> <td>R-Blank Limit</td> <td>2500.0 mAbs</td> </tr> <tr> <td>Cal Repts Range</td> <td>20.0 %</td> </tr> <tr> <td>Min Cal Repts</td> <td>50.0 mAbs</td> </tr> <tr> <td>Factor Change</td> <td>20 %</td> </tr> <tr> <td>M-Point Curve Fit</td> <td>0.0 %</td> </tr> </tbody> </table>		R-Blank Limit	2500.0 mAbs	Cal Repts Range	20.0 %	Min Cal Repts	50.0 mAbs	Factor Change	20 %	M-Point Curve Fit	0.0 %
Method	Curve Type																		
1-point	Linear																		
R-Blank Limit	2500.0 mAbs																		
Cal Repts Range	20.0 %																		
Min Cal Repts	50.0 mAbs																		
Factor Change	20 %																		
M-Point Curve Fit	0.0 %																		
<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> </tr> <tr> <th>No</th> <th>Stability</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>days</td> </tr> </tbody> </table>		Repeats		No	Stability	2	days												
Repeats																			
No	Stability																		
2	days																		
<table border="1"> <thead> <tr> <th colspan="2">Calibrator</th> </tr> <tr> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>TruCalU **</td> </tr> <tr> <td>2</td> <td></td> </tr> <tr> <td>3</td> <td></td> </tr> <tr> <td>4</td> <td></td> </tr> <tr> <td>5</td> <td></td> </tr> </tbody> </table>		Calibrator		Calibrator	Conc	1	TruCalU **	2		3		4		5					
Calibrator																			
Calibrator	Conc																		
1	TruCalU **																		
2																			
3																			
4																			
5																			

# АЛТ (аланинаминотрансфераза)

*IFCC, without P5P (метод без перидоксальфосфата)*



\* -Рекомендуемый параметр.

\*\* - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU) или по фактору (см инструкцию).

Контроль по TruLab N и TruLab P.