

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN ApoB Test Code: IL_ ApoB-d

Buttons: Measure, Sample Reagent, Ranges Limits, Calibration

Sample Type: Serum

Reaction Cycle: Standard (selected), Extended

Reporting Unit: г/л * (selected), User Define, Conv. Factor: 0.000

Decimal Points: 3

Methodology: Type: End Point (selected), Rate

Measuring Point: 16 - 33 (checked)

Photometric: 1 Wavelength, 2 Wavelength (selected)

Primary: 340, Secondary: 750

Correction Constant: Slope: 1.000, Intercept: 0.000

Test Name: NN ApoB Test Code: IL_ ApoB-d

Buttons: Measure, Sample Reagent, Ranges Limits, Calibration

Sample Volume

	Sample		Dilution	
	Volume	Sample Vol.	Diluent Vol.	
1	2.0	0.0	0.0	
2	2.0	30.0	60.0	
3	4.0	0.0	0.0	
4	0.0	0.0	0.0	

Diluent: [Dropdown]

Diluent Warning Limit: 0 Tests

Reagent Volume

R1: ApoB-1 R2: ApoB-2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	200	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	40	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

Condition No.	1	2	3	4
First Run	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Samp.Vol.Reduction	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Below N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Above N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic H	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
User Range L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> User Range H	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Noise	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Prozone	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> HIGH!	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> ABS!	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

Ranges			Limits																																
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0.8</td> <td>1.55</td> </tr> <tr> <td>Female</td> <td>0.75</td> <td>1.5</td> </tr> <tr> <td>Other</td> <td>0.75</td> <td>1.55</td> </tr> </tbody> </table>			Normal Range				Lower	Upper	Male	0.8	1.55	Female	0.75	1.5	Other	0.75	1.55	<table border="1"> <thead> <tr> <th colspan="2">Reaction Slope</th> <th colspan="2">Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/></td> <td>Negative</td> <td><input checked="" type="radio"/></td> <td>Positive</td> </tr> <tr> <td><input checked="" type="radio"/></td> <td>Above</td> <td><input type="radio"/></td> <td>Below</td> </tr> <tr> <td colspan="2"></td> <td>2600.0</td> <td>mAbs</td> </tr> </tbody> </table>		Reaction Slope		Absorbance Limit		<input type="radio"/>	Negative	<input checked="" type="radio"/>	Positive	<input checked="" type="radio"/>	Above	<input type="radio"/>	Below			2600.0	mAbs
Normal Range																																			
	Lower	Upper																																	
Male	0.8	1.55																																	
Female	0.75	1.5																																	
Other	0.75	1.55																																	
Reaction Slope		Absorbance Limit																																	
<input type="radio"/>	Negative	<input checked="" type="radio"/>	Positive																																
<input checked="" type="radio"/>	Above	<input type="radio"/>	Below																																
		2600.0	mAbs																																
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>			User Range		Lower	Upper			<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>100</td> <td>%</td> </tr> </tbody> </table>		Non-Linear Limit		100	%																					
User Range																																			
Lower	Upper																																		
Non-Linear Limit																																			
100	%																																		
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.003</td> <td>2.5</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	0.003	2.5	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0															
ValidRange																																			
Lower	Upper																																		
0.003	2.5																																		
Serum Index Limits																																			
	Limit																																		
Hemolysis	0.0																																		
Icterus	0.0																																		
Lipemia	0.0																																		
<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/></td> <td>Above</td> </tr> <tr> <td><input checked="" type="radio"/></td> <td>Below</td> </tr> <tr> <td>Limit</td> <td>Equation</td> </tr> <tr> <td>0.0</td> <td>none</td> </tr> <tr> <td>Judge Point</td> <td></td> </tr> <tr> <td>0</td> <td></td> </tr> </tbody> </table>			Prozone Limit		<input type="radio"/>	Above	<input checked="" type="radio"/>	Below	Limit	Equation	0.0	none	Judge Point		0		<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/></td> <td>On</td> </tr> <tr> <td><input checked="" type="radio"/></td> <td>Off</td> </tr> </tbody> </table>		Qualitative		<input type="radio"/>	On	<input checked="" type="radio"/>	Off											
Prozone Limit																																			
<input type="radio"/>	Above																																		
<input checked="" type="radio"/>	Below																																		
Limit	Equation																																		
0.0	none																																		
Judge Point																																			
0																																			
Qualitative																																			
<input type="radio"/>	On																																		
<input checked="" type="radio"/>	Off																																		

Test Name		Test Code		Measure	Sample Reagent	Ranges Limits	Calibration																																												
NN	ApoB	IL_	ApoB-d																																																
Calibration		Condition			Calibration/R-Blank Limit																																														
<table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td>M-point</td> <td></td> <td>5P Log</td> <td></td> </tr> <tr> <th colspan="2">Repeats</th> <th colspan="2">Stability</th> </tr> <tr> <td>2</td> <td></td> <td></td> <td>days</td> </tr> <tr> <th colspan="2">No</th> <td colspan="2"></td> </tr> <tr> <td>6</td> <td></td> <td colspan="2"></td> </tr> </tbody> </table>		Method		Curve Type		M-point		5P Log		Repeats		Stability		2			days	No				6				<input type="checkbox"/> Reagent Blank <input type="checkbox"/> Auto Reagent Blank by Bottle <input type="checkbox"/> Ask for calibration when reagent lot changes			<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td>mAbs</td> </tr> <tr> <th colspan="2">Cal Repts Range</th> </tr> <tr> <td>15.0</td> <td>%</td> </tr> <tr> <th colspan="2">Min Cal Repts</th> </tr> <tr> <td>100.0</td> <td>mAbs</td> </tr> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>35</td> <td>%</td> </tr> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </tbody> </table>			R-Blank Limit		2500.0	mAbs	Cal Repts Range		15.0	%	Min Cal Repts		100.0	mAbs	Factor Change		35	%	M-Point Curve Fit		0.0	%
Method		Curve Type																																																	
M-point		5P Log																																																	
Repeats		Stability																																																	
2			days																																																
No																																																			
6																																																			
R-Blank Limit																																																			
2500.0	mAbs																																																		
Cal Repts Range																																																			
15.0	%																																																		
Min Cal Repts																																																			
100.0	mAbs																																																		
Factor Change																																																			
35	%																																																		
M-Point Curve Fit																																																			
0.0	%																																																		
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>NaCl*</td> <td>0.0</td> </tr> <tr> <td>2</td> <td>TC-Apo-1</td> <td>**</td> </tr> <tr> <td>3</td> <td>TC-Apo-2</td> <td>**</td> </tr> <tr> <td>4</td> <td>TC-Apo-3</td> <td>**</td> </tr> <tr> <td>5</td> <td>TC-Apo-4</td> <td>**</td> </tr> <tr> <td></td> <td>TC-Apo-5</td> <td>**</td> </tr> </tbody> </table>								Calibrator				Calibrator	Conc	1	NaCl*	0.0	2	TC-Apo-1	**	3	TC-Apo-2	**	4	TC-Apo-3	**	5	TC-Apo-4	**		TC-Apo-5	**																				
Calibrator																																																			
	Calibrator	Conc																																																	
1	NaCl*	0.0																																																	
2	TC-Apo-1	**																																																	
3	TC-Apo-2	**																																																	
4	TC-Apo-3	**																																																	
5	TC-Apo-4	**																																																	
	TC-Apo-5	**																																																	

* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Калибровка многоточечная (6 точек, 0+5 уровней) Spline или 5P Log,
(калибратор TruCal ApoA1/B).

Приготовление серии калибраторов смотрите в инструкции к калибратору

Контроль TruLab L (Lipid) уровень 1 и 2.