

Антистрептолизин-О (ASLO)

Итипу TD (Иммунотурбидиметрический с латексными частицами)



Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN ASLO Test Code: IL_ ASLO-d Measure Sample Reagent Ranges Limits Calibration

Sample Type: Serum

Reaction Cycle: Standard Extended

Reporting Unit: ME /мл * Decimal Points: 1 User Define Conv. Factor: 0.000

Methodology: Type: End Point Rate Measuring Point: 19 - 30 Photometric: 1 Wavelength 2 Wavelength Primary: 570 Secondary: Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN ASLO Test Code: IL_ ASLO-d Measure Sample Reagent Ranges Limits Calibration

Sample Volume:

	Sample Volume	Dilution
	Volume	Sample Vol. Diluent Vol.
1	2.0	0.0 0.0
2	2.0	30.0 60.0
3	4.0	0.0 0.0
4	0.0	0.0 0.0

 Diluent: Diluent Warning Limit: 0 Tests

Reagent Volume: R1 ASLO-d1 R2 ASLO-d2

	Volume	Diluent Vol.	stiring	Warning Limit(tests)	Stability(days)
R1	165	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	32.5	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition:

Condition No.	1	2	3	4
First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Below N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Above N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic H	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
User Range L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Noise	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Prozone	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Антистрептолизин-О (ASLO)

Итипно TD (Иммунотурбидиметрический с латексными частицами)



Ranges			Limits																																																																																				
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0.0</td> <td>200.0</td> </tr> <tr> <td>Female</td> <td>0.0</td> <td>200.0</td> </tr> <tr> <td>Other</td> <td>0.0</td> <td>200.0</td> </tr> </tbody> </table>			Normal Range				Lower	Upper	Male	0.0	200.0	Female	0.0	200.0	Other	0.0	200.0	<table border="1"> <thead> <tr> <th>Reaction Slope</th> <th>Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Negative <input checked="" type="radio"/> Positive </td> <td> <input checked="" type="radio"/> Above <input type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>100 %</td> </tr> </tbody> </table> </td> <td> <table border="1"> <tbody> <tr> <td>2600.0 mAbs</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>5</td> <td>800</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Valid Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>5</td> <td>800</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> </tbody> </table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>100 %</td> </tr> </tbody> </table>	Non-Linear Limit	100 %	<table border="1"> <tbody> <tr> <td>2600.0 mAbs</td> </tr> </tbody> </table>	2600.0 mAbs	<table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0	<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>5</td> <td>800</td> </tr> </tbody> </table>			User Range		Lower	Upper	5	800			<table border="1"> <thead> <tr> <th colspan="2">Valid Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>5</td> <td>800</td> </tr> </tbody> </table>			Valid Range		Lower	Upper	5	800			<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>			Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0			<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>			Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off		
Normal Range																																																																																							
	Lower	Upper																																																																																					
Male	0.0	200.0																																																																																					
Female	0.0	200.0																																																																																					
Other	0.0	200.0																																																																																					
Reaction Slope	Absorbance Limit																																																																																						
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																																																																						
<table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>100 %</td> </tr> </tbody> </table>	Non-Linear Limit	100 %	<table border="1"> <tbody> <tr> <td>2600.0 mAbs</td> </tr> </tbody> </table>	2600.0 mAbs																																																																																			
Non-Linear Limit																																																																																							
100 %																																																																																							
2600.0 mAbs																																																																																							
<table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0																																																																											
Prozone Limit																																																																																							
<input type="radio"/> Above <input checked="" type="radio"/> Below																																																																																							
<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none																																																																																			
Limit	Equation																																																																																						
0.0	none																																																																																						
<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0																																																																																				
Judge Point																																																																																							
0																																																																																							
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>5</td> <td>800</td> </tr> </tbody> </table>			User Range		Lower	Upper	5	800																																																																															
User Range																																																																																							
Lower	Upper																																																																																						
5	800																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">Valid Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>5</td> <td>800</td> </tr> </tbody> </table>			Valid Range		Lower	Upper	5	800																																																																															
Valid Range																																																																																							
Lower	Upper																																																																																						
5	800																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>			Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																																																											
Serum Index Limits																																																																																							
	Limit																																																																																						
Hemolysis	0.0																																																																																						
Icterus	0.0																																																																																						
Lipemia	0.0																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>			Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																	
Qualitative																																																																																							
<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																						

Test Name	Test Code	Measure	Sample Reagent	Ranges Limits	Calibration																																		
NN ASLO	IL_ ASLO-d																																						
<table border="1"> <thead> <tr> <th colspan="2">Calibration</th> </tr> </thead> <tbody> <tr> <td>Method</td> <td>Curve Type</td> </tr> <tr> <td>M-point</td> <td>5P Log</td> </tr> <tr> <td>Repeats</td> <td>Stability</td> </tr> <tr> <td>2</td> <td>days</td> </tr> <tr> <td>No</td> <td></td> </tr> <tr> <td>6</td> <td></td> </tr> </tbody> </table>		Calibration		Method	Curve Type	M-point	5P Log	Repeats	Stability	2	days	No		6		<table border="1"> <thead> <tr> <th colspan="2">Condition</th> </tr> </thead> <tbody> <tr> <td><input type="checkbox"/></td> <td>Reagent Blank</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Auto Reagent Blank by Bottle</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Ask for calibration when reagent lot changes</td> </tr> </tbody> </table>		Condition		<input type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes	<table border="1"> <thead> <tr> <th colspan="2">Calibration/R-Blank Limit</th> </tr> </thead> <tbody> <tr> <td>R-Blank Limit</td> <td>2500.0 mAbs</td> </tr> <tr> <td>Cal Reps Range</td> <td>20.0 %</td> </tr> <tr> <td>Min Cal Reps</td> <td>100.0 mAbs</td> </tr> <tr> <td>Factor Change</td> <td>35 %</td> </tr> <tr> <td>M-Point Curve Fit</td> <td>0.0 %</td> </tr> </tbody> </table>		Calibration/R-Blank Limit		R-Blank Limit	2500.0 mAbs	Cal Reps Range	20.0 %	Min Cal Reps	100.0 mAbs	Factor Change	35 %	M-Point Curve Fit	0.0 %
Calibration																																							
Method	Curve Type																																						
M-point	5P Log																																						
Repeats	Stability																																						
2	days																																						
No																																							
6																																							
Condition																																							
<input type="checkbox"/>	Reagent Blank																																						
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																						
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																						
Calibration/R-Blank Limit																																							
R-Blank Limit	2500.0 mAbs																																						
Cal Reps Range	20.0 %																																						
Min Cal Reps	100.0 mAbs																																						
Factor Change	35 %																																						
M-Point Curve Fit	0.0 %																																						
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>NaCl-0.9</td> <td>0.0</td> </tr> <tr> <td>2</td> <td>TC-ASLO-1</td> <td>**</td> </tr> <tr> <td>3</td> <td>TC-ASLO-2</td> <td>**</td> </tr> <tr> <td>4</td> <td>TC-ASLO-3</td> <td>**</td> </tr> <tr> <td>5</td> <td>TC-ASLO-4</td> <td>**</td> </tr> <tr> <td>6</td> <td>TC-ASLO-5</td> <td>**</td> </tr> </tbody> </table>		Calibrator				Calibrator	Conc	1	NaCl-0.9	0.0	2	TC-ASLO-1	**	3	TC-ASLO-2	**	4	TC-ASLO-3	**	5	TC-ASLO-4	**	6	TC-ASLO-5	**														
Calibrator																																							
	Calibrator	Conc																																					
1	NaCl-0.9	0.0																																					
2	TC-ASLO-1	**																																					
3	TC-ASLO-2	**																																					
4	TC-ASLO-3	**																																					
5	TC-ASLO-4	**																																					
6	TC-ASLO-5	**																																					

Антистрептолизин-О (ASLO)

ИттипоTD (Иммунотурбидиметрический с латексными частицами)



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Калибровка многоточечная (6 точек, 0 + 5 ур.) Spline или 5P Log, калибратор TruCal ASO.

В качестве нулевой точки используйте физраствор.

Контроль TruLab Protein уровень 1 и 2.