

АСТ (аспартатаминотрансфераза)

IFCC, without P5P (метод без перидоксальфосфата)



Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **AST** Test Code: IL_ **AST-d** Measure Sample Reagent Ranges Limits Calibration

Sample Type: **Serum**

Reaction Cycle: Standard Extended

Reporting Unit: Ед/л * User Define
 Decimal Points: 1
 Conv. Factor: 0.000

Methodology:
 Type: End Point Rate
 Measuring Point: 19 - 29
 Photometric: 1 Wavelength 2 Wavelength
 Primary: 340 Secondary: 546
 Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN **AST** Test Code: IL_ **AST-d** Measure Sample Reagent Ranges Limits Calibration

Sample Volume

	Sample		Dilution	
	Volume	Sample Vol.	Diluent Vol.	
1	14.0	0.0	0.0	
2	2.0	0.0	0.0	
3	25.0	0.0	0.0	
4	10.0	5.0	95.0	

Diluent:
 Diluent Warning Limit: 0 Tests

Reagent Volume

R1: **AST-d1** R2: **AST-d2**

	Volume	Diluent Vol.	stiring	Warning Limit(tests)	Stability(days)
R1	140	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	35	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

Condition No.	1	2	3	4
First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Below N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Above N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic H	<input type="checkbox"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
User Range L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> User Range H	<input type="checkbox"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Noise	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Prozone	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> HIGH!	<input type="checkbox"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> ABS!	<input type="checkbox"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

АСТ (аспартатаминотрансфераза)

IFCC, without P5P (метод без перидоксальфосфата)

Ranges			Limits																																																																																				
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0</td> <td>35</td> </tr> <tr> <td>Female</td> <td>0</td> <td>31</td> </tr> <tr> <td>Other</td> <td>0</td> <td>35</td> </tr> </tbody> </table>			Normal Range				Lower	Upper	Male	0	35	Female	0	31	Other	0	35	<table border="1"> <thead> <tr> <th>Reaction Slope</th> <th>Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td> <input checked="" type="radio"/> Negative <input type="radio"/> Positive </td> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>25 %</td> </tr> </tbody> </table> </td> <td> <table border="1"> <tbody> <tr> <td>700.0 mAbs</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>700</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>700</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> </tbody> </table>		Reaction Slope	Absorbance Limit	<input checked="" type="radio"/> Negative <input type="radio"/> Positive	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>25 %</td> </tr> </tbody> </table>	Non-Linear Limit	25 %	<table border="1"> <tbody> <tr> <td>700.0 mAbs</td> </tr> </tbody> </table>	700.0 mAbs	<table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0	<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>700</td> </tr> </tbody> </table>			User Range		Lower	Upper	2	700			<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>700</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	2	700			<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>			Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0			<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>			Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off		
Normal Range																																																																																							
	Lower	Upper																																																																																					
Male	0	35																																																																																					
Female	0	31																																																																																					
Other	0	35																																																																																					
Reaction Slope	Absorbance Limit																																																																																						
<input checked="" type="radio"/> Negative <input type="radio"/> Positive	<input type="radio"/> Above <input checked="" type="radio"/> Below																																																																																						
<table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>25 %</td> </tr> </tbody> </table>	Non-Linear Limit	25 %	<table border="1"> <tbody> <tr> <td>700.0 mAbs</td> </tr> </tbody> </table>	700.0 mAbs																																																																																			
Non-Linear Limit																																																																																							
25 %																																																																																							
700.0 mAbs																																																																																							
<table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0																																																																											
Prozone Limit																																																																																							
<input type="radio"/> Above <input checked="" type="radio"/> Below																																																																																							
<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none																																																																																			
Limit	Equation																																																																																						
0.0	none																																																																																						
<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0																																																																																				
Judge Point																																																																																							
0																																																																																							
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>700</td> </tr> </tbody> </table>			User Range		Lower	Upper	2	700																																																																															
User Range																																																																																							
Lower	Upper																																																																																						
2	700																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>700</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	2	700																																																																															
ValidRange																																																																																							
Lower	Upper																																																																																						
2	700																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>			Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																																																											
Serum Index Limits																																																																																							
	Limit																																																																																						
Hemolysis	0.0																																																																																						
Icterus	0.0																																																																																						
Lipemia	0.0																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>			Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																	
Qualitative																																																																																							
<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																						

Test Name	Test Code	Measure	Sample Reagent	Ranges Limits	Calibration																																										
NN	AST	IL_	AST-d																																												
<table border="1"> <thead> <tr> <th colspan="2">Calibration</th> </tr> </thead> <tbody> <tr> <td>Method</td> <td>Curve Type</td> </tr> <tr> <td>1-point</td> <td>Linear</td> </tr> <tr> <td>Repeats</td> <td>Stability</td> </tr> <tr> <td>2</td> <td>days</td> </tr> <tr> <td>No</td> <td></td> </tr> </tbody> </table>		Calibration		Method	Curve Type	1-point	Linear	Repeats	Stability	2	days	No		<table border="1"> <thead> <tr> <th colspan="2">Condition</th> </tr> </thead> <tbody> <tr> <td><input checked="" type="checkbox"/></td> <td>Reagent Blank</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Auto Reagent Blank by Bottle</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Ask for calibration when reagent lot changes</td> </tr> </tbody> </table>		Condition		<input checked="" type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes	<table border="1"> <thead> <tr> <th colspan="2">Calibration/R-Blank Limit</th> </tr> </thead> <tbody> <tr> <td>R-Blank Limit</td> <td></td> </tr> <tr> <td>2500.0 mAbs</td> <td></td> </tr> <tr> <td>Cal Reps Range</td> <td></td> </tr> <tr> <td>20.0 %</td> <td></td> </tr> <tr> <td>Min Cal Reps</td> <td></td> </tr> <tr> <td>50.0 mAbs</td> <td></td> </tr> <tr> <td>Factor Change</td> <td></td> </tr> <tr> <td>20 %</td> <td></td> </tr> <tr> <td>M-Point Curve Fit</td> <td></td> </tr> <tr> <td>0.0 %</td> <td></td> </tr> </tbody> </table>		Calibration/R-Blank Limit		R-Blank Limit		2500.0 mAbs		Cal Reps Range		20.0 %		Min Cal Reps		50.0 mAbs		Factor Change		20 %		M-Point Curve Fit		0.0 %	
Calibration																																															
Method	Curve Type																																														
1-point	Linear																																														
Repeats	Stability																																														
2	days																																														
No																																															
Condition																																															
<input checked="" type="checkbox"/>	Reagent Blank																																														
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																														
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																														
Calibration/R-Blank Limit																																															
R-Blank Limit																																															
2500.0 mAbs																																															
Cal Reps Range																																															
20.0 %																																															
Min Cal Reps																																															
50.0 mAbs																																															
Factor Change																																															
20 %																																															
M-Point Curve Fit																																															
0.0 %																																															
<table border="1"> <thead> <tr> <th colspan="2">Calibrator</th> </tr> <tr> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>TruCalU **</td> </tr> <tr> <td>2</td> <td></td> </tr> <tr> <td>3</td> <td></td> </tr> <tr> <td>4</td> <td></td> </tr> <tr> <td>5</td> <td></td> </tr> </tbody> </table>		Calibrator		Calibrator	Conc	1	TruCalU **	2		3		4		5																																	
Calibrator																																															
Calibrator	Conc																																														
1	TruCalU **																																														
2																																															
3																																															
4																																															
5																																															

АСТ (аспартатаминотрансфераза)

IFCC, without P5P (метод без перидоксальфосфата)



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU) или по фактору (см инструкцию).

Контроль по TruLab N и TruLab P.