

Параметры для ввода в программу анализатора Beckman-Coulter AU 480

Parameters		Specific Test Parameters			
General	LIH	ISE	HbA1c	Calculated Test	Range
Test Name: <input type="text" value="Ca-p"/>		Type: <input type="text" value="Serum"/>		Operation: <input type="text" value="Yes"/>	
Sample Volume	<input type="text" value="1"/>	μL	Dilution	<input type="text" value="0"/>	μL
Pre-Dilution Rate	<input type="text" value="1"/>		OD Limit	<input type="text" value="-2.0"/>	<input type="text" value="2.5"/>
Rgt. Volume	R1(R1-1)	<input type="text" value="100"/>	μL	Dilution	<input type="text" value="0"/>
	R2(R2-1)	<input type="text" value="25"/>	μL	Dilution	<input type="text" value="0"/>
Wavelength	Pri	<input type="text" value="660"/>	nm	Sec.	<input type="text" value="800"/>
Method		<input type="text" value="END"/>			
Reaction Slope		<input type="text" value="+"/>			
Measuring Point1	First	<input type="text" value="0"/>	Last	<input type="text" value="27"/>	
Measuring Point2	First	<input type="text" value="0"/>	Last	<input type="text" value="10"/>	
Linearity Limit		<input type="text" value=""/>	%		
Lag Time Check		<input type="text" value=""/>			

Parameters		Specific Test Parameters			
General	LIH	ISE	HbA1c	Calculated Test	Range
Test Name: <input type="text" value="Ca-p"/>		Type: <input type="text" value="Serum"/>			
Value/Flag:	<input type="text" value=""/>	Level L:	<input type="text" value=""/>	Level H:	<input type="text" value=""/>
Specific Ranges:					
	Sex	Year	Month	Year	Month
<input type="checkbox"/>	1.	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>
<input type="checkbox"/>	2.	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>
<input type="checkbox"/>	3.	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>
<input type="checkbox"/>	4.	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>
<input type="checkbox"/>	5.	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>
<input type="checkbox"/>	6.	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>
<input type="checkbox"/>	7. No demographics			<input type="text" value="2.15"/>	<input type="text" value="2.57"/>
<input type="checkbox"/>	8. Not within expected values			<input type="text" value="2.15"/>	<input type="text" value="2.57"/>
Unit	<input type="text" value="mmol/L"/>		Decimal Places	<input type="text" value="2"/>	
Panic Value					
		Low	<input type="text" value=""/>	High	<input type="text" value=""/>

Parameters		Calibration Parameters			
Calibrators	Calibration Specific	STAT Table Calibration			
General	ISE				
Test Name: <input type="text" value="Ca-p"/>		Type: <input type="text" value="Serum"/>		Use Serum Cal. <input type="checkbox"/>	
Calibration Type:	<input type="text" value="AB"/>	Formula:	<input type="text" value="Y=AX+B"/>		Counts: <input type="text" value="2"/>
<Calibrator Parameters>					
Calibrator	OD	Conc	Factor/OD range	Low	High
Point 1:	<input type="text" value="TruCal U"/>	<input type="text" value="*"/>	<input type="text" value="-99999"/>	<input type="text" value="99999"/>	
Point 2:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	
Point 3:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	
Point 4:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	
Point 5:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	
Point 6:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	
Point 7:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	
Point 8:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	
Point 9:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	
Point 10:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	
Slope Check: <input type="text" value="None"/>					
Allowance Range Check					
		<input type="checkbox"/>	Reagent Blank	<input type="text" value=""/>	
		<input type="checkbox"/>	Calibration	<input type="text" value=""/>	
Advanced Calibration Operation: <input type="text" value=""/>					
Interval (RB/ACAL): <input type="text" value=""/>					
<Point Cal. For Master Curve>					
		No. of Correction Points	<input type="text" value=""/>	Use Master Curve	<input type="checkbox"/>
				<input type="checkbox"/>	Lot Calibration
<Calibrator Parameters>					
Calibrator	OD	Conc	Low	High	Stability
Point 1:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	Reagent Blank
Point 2:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>	Day
					Hour
					Day
					Hour
MB Type Factor: <input type="text" value=""/>					
		1-Point Calibration Point	<input type="checkbox"/>	<input type="checkbox"/>	
			<input type="checkbox"/>	with Conc-0	

Диапазон нормальных значений указан в соответствии с рекомендациями производителя реагентов к набору и может быть изменен пользователем в соответствии с местными требованиями.

Калибровка линейная, АВ (калибратор TruCal U, или стандарт из набора).

* -вводится из паспорта к калибратору (стандарт)

Контроль TruLab N и TruLab P.