

## Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN ApoA1    Test Code: IL\_ ApoA1-d    Measure    Sample Reagent    Ranges Limits    Calibration

Sample Type: **Serum**

Reaction Cycle:  Standard     Extended

Reporting Unit:  г/л \*    Decimal Points: 3     User Define    Conv. Factor: 0.000

Methodology: Type:  End Point     Rate

Measuring Point: 16 -  33

Photometric:  1 Wavelength     2 Wavelength

Primary: 570    Secondary: [ ]

Correction Constant: Slope: 1.000    Intercept: 0.000

Test Name: NN ApoA1    Test Code: IL\_ ApoA1-d    Measure    **Sample Reagent**    Ranges Limits    Calibration

Sample Volume

	Sample		Dilution	
	Volume	Sample Vol.	Diluent Vol.	
1	2.0	0.0	0.0	
2	2.0	30.0	60.0	
3	4.0	0.0	0.0	
4	0.0	0.0	0.0	

Diluent: [ ]    Diluent Warning Limit: 0 Tests

Reagent Volume

R1: ApoA-1    R2: ApoA-2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	250	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	50	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

Condition No.	1	2	3	4
First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Below N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Above N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic H	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
User Range L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> User Range H	<input type="checkbox"/>	<input checked="" type="radio"/>	<input type="checkbox"/>	<input type="checkbox"/>
Noise	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Prozone	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> HIGH!	<input type="checkbox"/>	<input checked="" type="radio"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> ABS!	<input type="checkbox"/>	<input checked="" type="radio"/>	<input type="checkbox"/>	<input type="checkbox"/>

# Аполипопротеин А1

ИммуноТD (Иммунотурбидиметрический)

Ranges			Limits																																																																																		
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>1.1</td> <td>1.7</td> </tr> <tr> <td>Female</td> <td>1.2</td> <td>1.9</td> </tr> <tr> <td>Other</td> <td>1.1</td> <td>1.9</td> </tr> </tbody> </table>			Normal Range				Lower	Upper	Male	1.1	1.7	Female	1.2	1.9	Other	1.1	1.9	<table border="1"> <thead> <tr> <th colspan="2">Reaction Slope</th> <th colspan="2">Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> Negative</td> <td><input checked="" type="radio"/> Positive</td> <td><input checked="" type="radio"/> Above</td> <td><input type="radio"/> Below</td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>100</td> <td></td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <th>Value</th> <th></th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.002</td> <td>2.5</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="3"> <p>Qualitative</p> <p><input type="radio"/> On <input checked="" type="radio"/> Off</p> </td> <td colspan="2"></td> </tr> </tbody> </table>		Reaction Slope		Absorbance Limit		<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>100</td> <td></td> </tr> </tbody> </table>		Non-Linear Limit		Value	%	100		<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>		Prozone Limit		Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <th>Value</th> <th></th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table>		Judge Point		Value		0		<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>			User Range		Lower	Upper			<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.002</td> <td>2.5</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	0.002	2.5	<p>Qualitative</p> <p><input type="radio"/> On <input checked="" type="radio"/> Off</p>				
Normal Range																																																																																					
	Lower	Upper																																																																																			
Male	1.1	1.7																																																																																			
Female	1.2	1.9																																																																																			
Other	1.1	1.9																																																																																			
Reaction Slope		Absorbance Limit																																																																																			
<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below																																																																																		
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>100</td> <td></td> </tr> </tbody> </table>		Non-Linear Limit		Value	%	100		<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>		Prozone Limit		Limit	Equation	0.0	none																																																																						
Non-Linear Limit																																																																																					
Value	%																																																																																				
100																																																																																					
Prozone Limit																																																																																					
Limit	Equation																																																																																				
0.0	none																																																																																				
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <th>Value</th> <th></th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table>		Judge Point		Value		0																																																																			
Serum Index Limits																																																																																					
	Limit																																																																																				
Hemolysis	0.0																																																																																				
Icterus	0.0																																																																																				
Lipemia	0.0																																																																																				
Judge Point																																																																																					
Value																																																																																					
0																																																																																					
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>			User Range		Lower	Upper			<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.002</td> <td>2.5</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	0.002	2.5																																																																					
User Range																																																																																					
Lower	Upper																																																																																				
ValidRange																																																																																					
Lower	Upper																																																																																				
0.002	2.5																																																																																				
<p>Qualitative</p> <p><input type="radio"/> On <input checked="" type="radio"/> Off</p>																																																																																					

Test Name		Test Code		Measure	Sample Reagent	Ranges Limits	Calibration																						
NN	ApoA1	IL_	ApoA1-d																										
Calibration		Condition			Calibration/R-Blank Limit																								
<table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td>M-point</td> <td></td> <td>5P Log</td> <td></td> </tr> </tbody> </table>		Method		Curve Type		M-point		5P Log		<p><input type="checkbox"/> Reagent Blank</p> <p><input type="checkbox"/> Auto Reagent Blank by Bottle</p> <p><input type="checkbox"/> Ask for calibration when reagent lot changes</p>			<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> <tr> <th>Value</th> <th>mAbs</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td></td> </tr> </tbody> </table>			R-Blank Limit		Value	mAbs	2500.0									
Method		Curve Type																											
M-point		5P Log																											
R-Blank Limit																													
Value	mAbs																												
2500.0																													
<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> <th colspan="2">Stability</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> <td></td> <td>days</td> </tr> </tbody> </table>		Repeats		Stability		2			days				<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>15.0</td> <td></td> </tr> </tbody> </table>			Cal Reps Range		Value	%	15.0									
Repeats		Stability																											
2			days																										
Cal Reps Range																													
Value	%																												
15.0																													
<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> <tr> <th>Value</th> <th></th> </tr> </thead> <tbody> <tr> <td>6</td> <td></td> </tr> </tbody> </table>		No		Value		6					<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <th>Value</th> <th>mAbs</th> </tr> </thead> <tbody> <tr> <td>100.0</td> <td></td> </tr> </tbody> </table>			Min Cal Reps		Value	mAbs	100.0											
No																													
Value																													
6																													
Min Cal Reps																													
Value	mAbs																												
100.0																													
<table border="1"> <thead> <tr> <th colspan="2">Calibrator</th> </tr> <tr> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1 NaCl*</td> <td>0.0</td> </tr> <tr> <td>2 TC-Apo-1</td> <td>**</td> </tr> <tr> <td>3 TC-Apo-2</td> <td>**</td> </tr> <tr> <td>4 TC-Apo-3</td> <td>**</td> </tr> <tr> <td>5 TC-Apo-4</td> <td>**</td> </tr> <tr> <td>TC-Apo-5</td> <td>**</td> </tr> </tbody> </table>		Calibrator		Calibrator	Conc	1 NaCl*	0.0	2 TC-Apo-1	**	3 TC-Apo-2	**	4 TC-Apo-3	**	5 TC-Apo-4	**	TC-Apo-5	**				<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>35</td> <td></td> </tr> </tbody> </table>			Factor Change		Value	%	35	
Calibrator																													
Calibrator	Conc																												
1 NaCl*	0.0																												
2 TC-Apo-1	**																												
3 TC-Apo-2	**																												
4 TC-Apo-3	**																												
5 TC-Apo-4	**																												
TC-Apo-5	**																												
Factor Change																													
Value	%																												
35																													
					<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td></td> </tr> </tbody> </table>			M-Point Curve Fit		Value	%	0.0																	
M-Point Curve Fit																													
Value	%																												
0.0																													

\* -Рекомендуемый параметр.

\*\* - Вводится из паспорта калибратору.

Калибровка многоточечная (6 точек, 0+5 уровней) Spline или 5P Log,  
(калибратор TruCal ApoA1/B).

Приготовление серии калибраторов смотрите в инструкции к калибратору

Контроль TruLab L (Lipid) уровень 1 и 2.