

Альбумин в моче, (микроальбумин)

ИтипоТD (Иммунотурбидиметрический)



Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **MkAlb** Test Code: IL_ **MkAlb-d** Measure Sample Reagent Ranges Limits Calibration

Sample Type: **Serum**

Reaction Cycle: Standard Extended

Reporting Unit: Мг/л * Decimal Points: 1 User Define Conv.Factor: 0.000

Methodology: Type: End Point Rate

Measuring Point: 16 - 33

Photometric: 1 Wavelength 2 Wavelength

Primary: 405 Secondary: 750

Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN **MkAlb** Test Code: IL_ **MkAlb-d** Measure **Sample Reagent** Ranges Limits Calibration

Sample Volume

	Sample		Dilution	
	Volume	Sample Vol.	Diluent Vol.	
1	7.5	0.0	0.0	
2	2.0	0.0	0.0	
3	10.0	0.0	0.0	
4	0.0	0.0	0.0	

Diluent: Diluent Warning Limit: 0 Tests

Reagent Volume

R1: **MAIb-1** R2: **MAIb-2**

	Volume	Diluent Vol.	stiring	Warning Limit(tests)	Stability(days)
R1	150	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	30	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

Condition No.	1	2	3	4
First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Below N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Above N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Panic H	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
User Range L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> User Range H	<input type="checkbox"/>	<input checked="" type="radio"/>	<input type="checkbox"/>	<input type="checkbox"/>
Noise	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Prozone	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> HIGH!	<input type="checkbox"/>	<input checked="" type="radio"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/> ABS!	<input type="checkbox"/>	<input checked="" type="radio"/>	<input type="checkbox"/>	<input type="checkbox"/>

Альбумин в моче, (микроальбумин)

ИттипоТD (Иммунотурбидиметрический)

Ranges			Limits																																																																																				
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0.0</td> <td>30</td> </tr> <tr> <td>Female</td> <td>0.0</td> <td>30</td> </tr> <tr> <td>Other</td> <td>0.0</td> <td>30</td> </tr> </tbody> </table>			Normal Range				Lower	Upper	Male	0.0	30	Female	0.0	30	Other	0.0	30	<table border="1"> <thead> <tr> <th>Reaction Slope</th> <th>Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Negative <input checked="" type="radio"/> Positive </td> <td> <input checked="" type="radio"/> Above <input type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>100 %</td> </tr> </tbody> </table> </td> <td> <table border="1"> <tbody> <tr> <td>2600.0 mAbs</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>3</td> <td>350</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>3</td> <td>350</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> </tbody> </table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>100 %</td> </tr> </tbody> </table>	Non-Linear Limit	100 %	<table border="1"> <tbody> <tr> <td>2600.0 mAbs</td> </tr> </tbody> </table>	2600.0 mAbs	<table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0	<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>3</td> <td>350</td> </tr> </tbody> </table>			User Range		Lower	Upper	3	350			<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>3</td> <td>350</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	3	350			<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>			Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0			<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>			Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off		
Normal Range																																																																																							
	Lower	Upper																																																																																					
Male	0.0	30																																																																																					
Female	0.0	30																																																																																					
Other	0.0	30																																																																																					
Reaction Slope	Absorbance Limit																																																																																						
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																																																																						
<table border="1"> <thead> <tr> <th>Non-Linear Limit</th> </tr> </thead> <tbody> <tr> <td>100 %</td> </tr> </tbody> </table>	Non-Linear Limit	100 %	<table border="1"> <tbody> <tr> <td>2600.0 mAbs</td> </tr> </tbody> </table>	2600.0 mAbs																																																																																			
Non-Linear Limit																																																																																							
100 %																																																																																							
2600.0 mAbs																																																																																							
<table border="1"> <thead> <tr> <th>Prozone Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit	<input type="radio"/> Above <input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0																																																																											
Prozone Limit																																																																																							
<input type="radio"/> Above <input checked="" type="radio"/> Below																																																																																							
<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none																																																																																			
Limit	Equation																																																																																						
0.0	none																																																																																						
<table border="1"> <thead> <tr> <th>Judge Point</th> </tr> </thead> <tbody> <tr> <td>0</td> </tr> </tbody> </table>		Judge Point	0																																																																																				
Judge Point																																																																																							
0																																																																																							
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>3</td> <td>350</td> </tr> </tbody> </table>			User Range		Lower	Upper	3	350																																																																															
User Range																																																																																							
Lower	Upper																																																																																						
3	350																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>3</td> <td>350</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	3	350																																																																															
ValidRange																																																																																							
Lower	Upper																																																																																						
3	350																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>			Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																																																											
Serum Index Limits																																																																																							
	Limit																																																																																						
Hemolysis	0.0																																																																																						
Icterus	0.0																																																																																						
Lipemia	0.0																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>			Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																	
Qualitative																																																																																							
<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																						

Test Name		Test Code		Measure	Sample Reagent	Ranges Limits	Calibration																								
NN	MkAlb	IL	MkAlb-d																												
Calibration		Condition		Calibration/R-Blank Limit																											
<table border="1"> <thead> <tr> <th>Method</th> <th>Curve Type</th> </tr> </thead> <tbody> <tr> <td>M-point</td> <td>5P Log</td> </tr> </tbody> </table>		Method	Curve Type	M-point	5P Log	<input type="checkbox"/> Reagent Blank <input type="checkbox"/> Auto Reagent Blank by Bottle <input type="checkbox"/> Ask for calibration when reagent lot changes		<table border="1"> <tbody> <tr> <td>R-Blank Limit</td> <td>2500.0 mAbs</td> </tr> <tr> <td>Cal Repts Range</td> <td>15.0 %</td> </tr> <tr> <td>Min Cal Repts</td> <td>200.0 mAbs</td> </tr> <tr> <td>Factor Change</td> <td>35 %</td> </tr> <tr> <td>M-Point Curve Fit</td> <td>0.0 %</td> </tr> </tbody> </table>				R-Blank Limit	2500.0 mAbs	Cal Repts Range	15.0 %	Min Cal Repts	200.0 mAbs	Factor Change	35 %	M-Point Curve Fit	0.0 %										
Method	Curve Type																														
M-point	5P Log																														
R-Blank Limit	2500.0 mAbs																														
Cal Repts Range	15.0 %																														
Min Cal Repts	200.0 mAbs																														
Factor Change	35 %																														
M-Point Curve Fit	0.0 %																														
<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> </tr> </tbody> </table>		Repeats		2																											
Repeats																															
2																															
<table border="1"> <thead> <tr> <th colspan="2">Stability</th> </tr> </thead> <tbody> <tr> <td></td> <td>days</td> </tr> </tbody> </table>		Stability			days																										
Stability																															
	days																														
<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> </thead> <tbody> <tr> <td>6</td> <td></td> </tr> </tbody> </table>		No		6																											
No																															
6																															
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>NaCl*</td> <td>0.0</td> </tr> <tr> <td>2</td> <td>TC-MAIb-1</td> <td>**</td> </tr> <tr> <td>3</td> <td>TC-MAIb-2</td> <td>**</td> </tr> <tr> <td>4</td> <td>TC-MAIb-3</td> <td>**</td> </tr> <tr> <td>5</td> <td>TC-MAIb-4</td> <td>**</td> </tr> <tr> <td>6</td> <td>TC-MAIb-5</td> <td>**</td> </tr> </tbody> </table>		Calibrator				Calibrator	Conc	1	NaCl*	0.0	2	TC-MAIb-1	**	3	TC-MAIb-2	**	4	TC-MAIb-3	**	5	TC-MAIb-4	**	6	TC-MAIb-5	**						
Calibrator																															
	Calibrator	Conc																													
1	NaCl*	0.0																													
2	TC-MAIb-1	**																													
3	TC-MAIb-2	**																													
4	TC-MAIb-3	**																													
5	TC-MAIb-4	**																													
6	TC-MAIb-5	**																													

Альбумин в моче, (микроальбумин)

ИттипотД (Иммунотурбидиметрический)



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Калибровка многоточечная (6 точек, 0+5 уровней) Spline или 5P Log,
калибратор TruCal Alb U CSF

Контроль TruLab Urine уровень 1 и 2.